



WalkMed






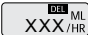

350VL Ambulatory Pumps

Patient Quick Reference Card

Keep this card in your carrying pouch or on your person. For complete guidance, refer to the instructions given to you by your health care provider.

Brief Instructions

To Start Medication Delivery:



1. Press the POWER  key to turn pump on. The pump will display .
2. Press the START  key to begin delivery.
The   light will start blinking green. The display will alternate between delivery rate  and volume delivered  screens.
3. Place front cover on pump.

To Stop Medication Delivery:

1. Press the STOP  key. The pump will display .

To Change the Battery:

Use a 9-volt alkaline battery. Duracell® MN 1604 recommended.

1. Press the STOP  key and turn pump off .
2. Remove battery. Wait approximately 15 seconds before installing new battery.
3. Follow **Start Medication Delivery** instructions above.

Additional Information:



WalkMed Infusion LLC
6555 South Kenton Street
Suite 304
Centennial, CO 80111, USA
303-420-9569

EC REP

Priory Analysts Ltd.
Mazars, The Pinnacle
160 Midsummer Boulevard
Milton Keynes, MK9 31FF
United Kingdom





Problem Solving Table

Software Revision Level 2.0 [.1-.9]

1. Match the message you see on your pump display with the **Display Message** in the left column below.
2. Follow the **Corrective Action** given in the right column.
3. If you cannot correct the problem following the **Corrective Action**, follow the directions given by your health care provider.

() _____ - _____

Display Message	Sounds/ Lights	Condition	Corrective Action
	Sounds: Slow beeping Light: Blinking yellow	Not delivering	Press key to start delivery. Press key to silence alert for 5 minutes.
	Sounds: Slow beeping Light: Blinking yellow	Low battery	Press key to start delivery. Replace battery within 30 minutes.
	Sounds: Continuous tone Light: Continuous red	Depleted battery	Remove depleted battery to silence alarm. Replace battery. Press key to turn pump on. Press key to restart.
	Sounds: Continuous tone Light: Blinking red	Clamp bar is open	Press key to silence alert. Close clamp bar. Press key to restart.
	Sounds: Continuous tone Light: Blinking red	IV tubing occlusion	Check tubing system for kinks and correct. Pump will resume delivery when line is cleared. Press key to silence alert. Check tubing system for kinks and correct. Press key to restart.
<p>↑ 1-8, I O or U will appear here.</p>	Sounds: Continuous tone Light: Continuous red	System malfunction	Write down Error message from display. Remove battery to silence alarm. Take pump out of service. Contact support with Error message.
scrolling 	Sounds: Slow beeping Light: Blinking yellow	Total volume limit reached	The pump will continue delivery at the KVO rate for approximately 4 hours An audible alert will beep every 30 minutes.
	Sounds: Fast beeping Light: Blinking red	End of KVO	Press key to silence alert. Clear END alert by: Reset volume delivered. - or - Increase total volume limit amount. - or - Set total volume limit to OFF.
		Failed key in keypad	Press key to turn pump off. Take pump out of service.
	Sounds: Continuous tone Light: Continuous red	System malfunction during battery install	Remove battery and wait 15 seconds before re-installing